


# TRAUS<sup>SUS10</sup>

## Connection of each part


### WARNING

Please make sure that power is off!  
(Otherwise, it might cause a malfunction.)

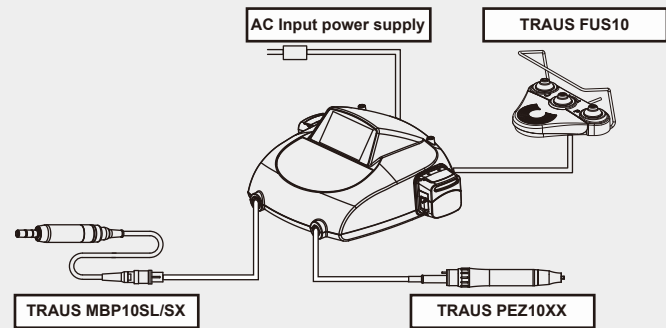

## Piezo tip connection

1. Insert & turn the piezo tip clockwise by hand.  
Then use the torque wrench to tighten.

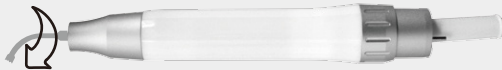

2. Make sure to tighten the tip fully using the torque wrench like the image below.



**CAUTION** Don't insert the torque wrench upside down.

### Piezo handpiece

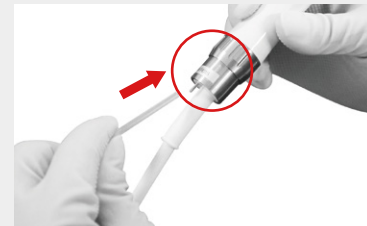


FIG (A)

Connect the irrigation tube to the irrigation pipe that is located at the end of the piezo handpiece cord as in FIG(A).



### CAUTION

3. Once the tip is placed correctly on the handpiece, press **Clean/Ready** button until the saline solution reached the tip.



A




B


## Irrigation tube set up




Open the irrigation pump roller cover upward.

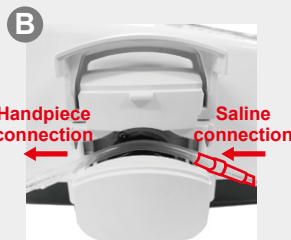

Insert the irrigation tube connectly into the center of irrigation pump.

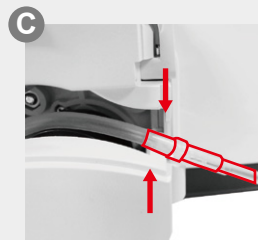

When inserting a tube, place the pump cover to close where the arrow is showing above.  
Otherwise, irrigation flow might not be efficient or will leak.

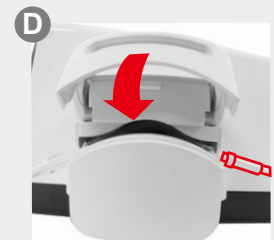

Close the irrigation pump cover downward.



When closing the pump roller cover make sure that the irrigation tube is positioned as shown in the figure.



Make sure the pump roller cover is completely closed.




Keep the pump cover open if it's not in use for a long time. Otherwise, irrigation tube might be defective and it can cause a malfunction. Please pay attention to the direction of the irrigation tube.



The tube provided is for single use only.


## Connection of each part

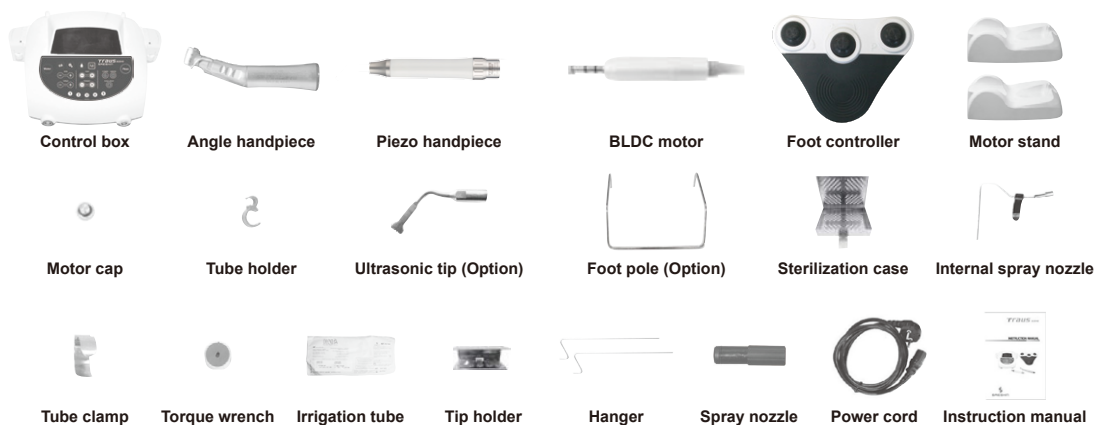

## How to maintain a dental handpieces


### 1 How to Clean

- Rotate the head in alcohol about 5-6 seconds with high speed right after surgery.
- After rotating the head, put the two dissembled parts in alcohol in order to remove any remains before they get coagulated as shown below.

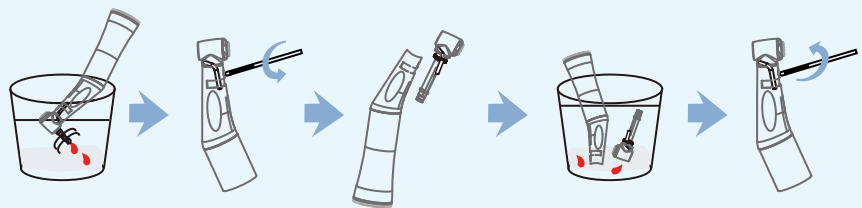

### 2 How to lubricate

- Please insert the oil spray into the assembled handpiece until clear liquid comes out. Refer to the below pic. for the direction of oiling.
- For the disassembled parts after alcohol cleaning, Oiling is available.
- Finish the cleaning of the oiled handpiece with air dry.

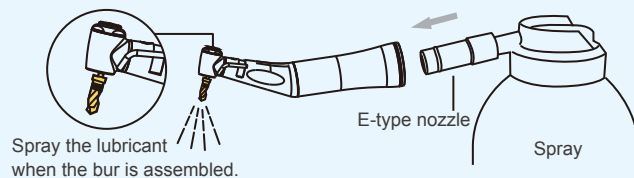

### 3 How to sterilize

#### Warning



- For following cases, it may cause serious damage to the product.
- It may cause breakdown, if the handpiece is kept with remains such as physiological saline, blood, its coagulation or rust. (Gear and cartridge may be damaged easily)
- Please refer to the instruction manual.